

## HT Vista – QA

- Start with checking the "shutdown button" functionality
  - See that the light left to the button work properly
  - Press the button for 10 seconds
    - Wait until the device finish the restart
  - Press a short press on the button
    - Press the restart button on the screen
    - Wait until the device finish the restart
  - Press a short press on the button
    - Press the shutdown button on the screen
    - Wait until the screen is black
  - Press a short press on the button
    - Wait until the device turned up
- Add patient -
  - Go to add new patient tab and fill the following information
    - full name - test
    - Patient ID - test123456
    - Breed - select any breed
    - Sex - select any sex
    - Age - select any age
  - Press Add button
- Check scanner LEDs and scanner button
  - Go to patients tab
  - Select the patient you've created
  - Press on scan new mass
  - Select a random spot on the dog
  - Press next
  - Wait until the calibration finish (about one minute)
  - Take out the scanner out of the dock
  - Place it on a piece of paper with handwriting on it
    - See that the camera have a good focus
  - Press scanner button
  - Watch the 3 LEDs on the scanner
    - They should all be working
    - They should be visible
    - They should have the same color
  - Finish the scan and wait for the results
    - You should see the result = 5
- System check
  - Go to settings page
  - Go to trouble shoot tab
  - Place the scanner on the dock
  - Press on perform in the system check
  - Press again perform
    - You should see that all 3 parameters has passed